


## Recommended Application Instructions for Allflex Electronic Identification Ear Tags


Apply Allflex Electronic ID Tags with the Universal Total Tagger (Red). Use the Red Blunt Pin and remove the Insert (white or black) from the base of the jaws.

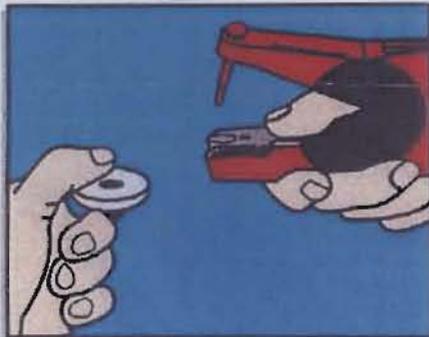

1. To load, depress spring clip and insert the female EID tag. Ensure that the raised portion of the tag, which encloses the transponder chip, is placed in the open portion of the jaws.



2. Slip the male tag completely onto the blunt applicator pin. Squeeze the jaws together lightly to ensure the male shaft is in line with the female.

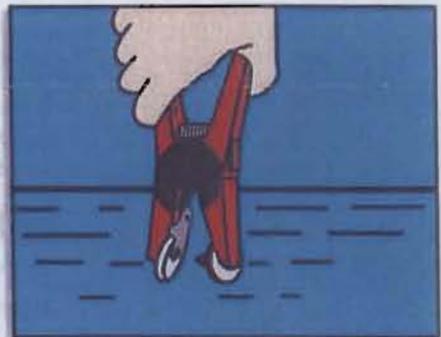

3. Dip the jaws of the applicator holding the tag into an antiseptic or disinfectant solution.

## Recommended EID Tag Placement for Cattle



Application site must be free of foreign debris prior to placement of tags on the animal. Review application instruction prior to tagging.

1. The EID tag should be placed in the middle of the ear between the two cartilage ribs close to the head.
2. The female portion of the tag should be on the inside of the ear with EID tag application. Note that this is a thicker part of the ear. Application may be more difficult than when applying a visual tag.
3. **IMPORTANT:** Caution, "Free Air Space" is critical for proper healing and retention. Inspect placement after tagging to ensure there is sufficient space between ear and EID tag.

[www.allflexusa.com](http://www.allflexusa.com)

All product offerings subject to change without further notice.

®/™ Trademarks registered or applied for of Allflex USA, Inc.  
© 2007 Allflex USA, Inc. AUI1 5842

Allflex USA, Inc.  
P.O. Box 612266  
Dallas/Ft. Worth Airport, TX 75261-2266  
(800) 989-TAGS (8247)  
Customer Toll Free FAX (877) 456-3969

Allflex Canada  
4135 Bérard  
St-Hyacinthe, QC J2S 8Z8  
(866) 505-TAGS (8247)  
FAX (450) 261-8028